

## SYN-200: INSTRUCTIONS FOR USE

### Rx Only

#### INDICATIONS FOR USE:

- SYN-200 is a sterile solution containing Sodium Hyaluronate (20mg/ml).
- Sterile Post-surgical lavage solution to replace synovial fluid lost during surgery.
- Helps normalize the joint after arthrotomy.
- For use on cats, dogs and horses.

#### DIRECTIONS FOR USE:

- Install the medical device as the final lavage prior to surgical closure of the joint space.
- Volume to be determined by veterinary surgeon.
- Discard any unused portion of vial contents.

#### WARNING:

- Not for food animal use.
- Keep out of reach of children.
- For veterinary use only.

#### STORAGE:

- Store in a cool, dry place.
- Do not freeze

#### MANUFACTURED FOR:

Novita Biotechnology (Pty) Ltd  
28 Brommaert Avenue  
Constantia  
7806  
[www.novita-biotech.com](http://www.novita-biotech.com)  
[info@novita-biotech.com](mailto:info@novita-biotech.com)  
SAHPRA: 00001494MD